

**GENTLE,
PRECISE AND
PAINLESS**

Sample collection
under the tongue

SARS-CoV-2 WATMIND GENTLE QUICK TEST **BELOW THE TONGUE**




FOR CHILDREN AND ADULTS



- Pleasant and stress-free for children
- No age restrictions
- Precise and highly sensitive
- No nasal swab
- No sucking or spitting
- Reliable even with virus mutations

95% SENSITIVITY

99% SPECIFICITY

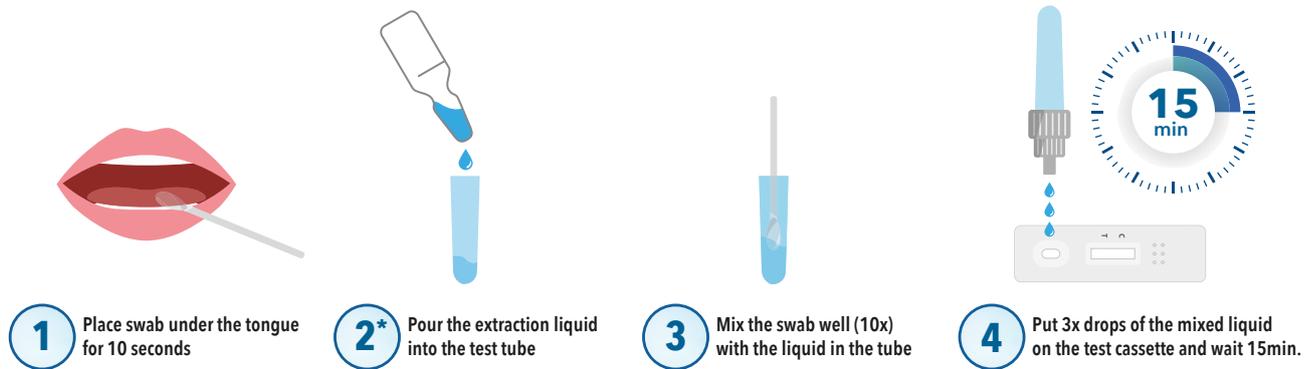

*Step 2 only for 1pc test kits (for reasons of transport safety). In the 5pcs test kits, the tubes are blister-packed and already pre-filled.

SARS-CoV-2 Watmind Quick Test

- Sample collection under the tongue
- No sucking or spitting
- No nasal swab
- Suitable for all age groups:
Young children, primary school children, adolescents and adults
- 95% sensitivity | 99% specificity
- Reliable even with mutations
- Evaluated by the Paul Ehrlich Institute in Germany
- Approved in Germany as a lay test
- Test result after 15 minutes
- Use on children under adult supervision
- Shelf life up to 18 months
- Storage at 2°-30°C
- Test to be performed at room temperature 18°-26°C
- 1 or 5 tests per packaging unit

1x Test PACKAGE CONTENT

- Specimen swab
- Test liquid
- Test tube
- SARS-CoV-2 Antigen test cassette
- User's instructions

5x Tests PACKAGE CONTENT

- 5x Specimen swab
- 5x Test tube with pre-filled test liquid
- 5x SARS-CoV-2 Antigen test cassette
- User's instructions



Clinical study*2

Ct Value	Samples Quantity	RT-PCR Results	Antigen Test in comparison to RT-PCR
≤ 30	46	positive	44 / 46 = 95,65%
≤ 36	157	positive	142 / 157 = 90,45%
> 40	182	negative	181 / 182 = 99,45%

Days after onset of symptoms	Samples Quantity	RT-PCR Results	Antigen Test in comparison to RT-PCR
≤ 7	89	positive	85 / 89 = 95,51%
≤ 14	116	positive	107 / 116 = 92,24%
> 14	41	positive	35 / 41 = 85,37%

*2Watmind SARS-CoV-2 Ag Quick Test. User's instructions, Art. Nr. LFA0401-5N